



# Influenza A & Influenza B Antigen Rapid Test Kit (colloidal gold method)


The Influenza A & Influenza B Antigen Rapid Test Kit (Colloidal Gold Method) is a lateral flow Immunology for qualitative detection of antigen that can specifically identify influenza A virus nucleoprotein and influenza B virus nucleoprotein to avoid detection of influenza virus nucleoprotein by chromatography. The operation steps are simple and test results are interpreted visually at 15-20 minutes based on the presence or absence of visually colored lines. The diagnosis can be achieved within 1-7 days after infection, which is of great significance for prognosis and targeted treatment.

## Test Procedure


**1** Influenza A & Influenza B Antigen Rapid Test Kit (Colloidal Gold Method)




**2** Unscrew the big lid of the sample extraction tube

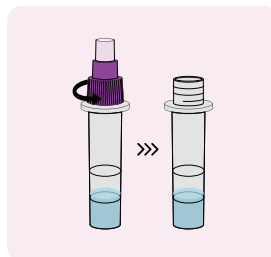

**3** Insert the sampled swab into the solution in the sample extraction tube

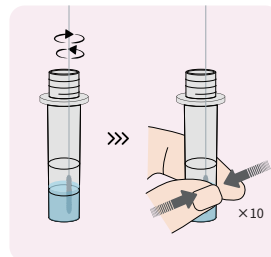


**4** Squeeze the villi part of the swab 5 times, then remove and discard the swab

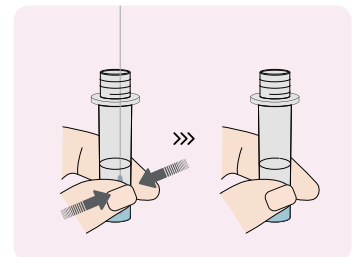


**5** Cover the dropper lid on the sample extraction tube after sampling




**6** Unscrew the little lid of the sample extraction tube

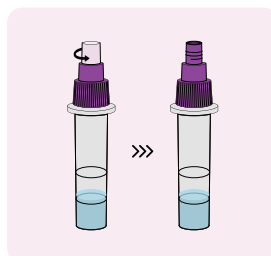


**7** Add 5 drops (approximately 100 μL) of the treated sample into the sample well of the test card

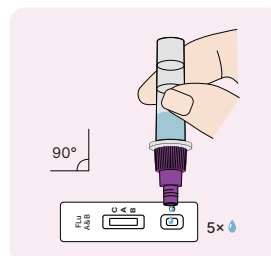

**8** The detection results of the kit are interpreted as above

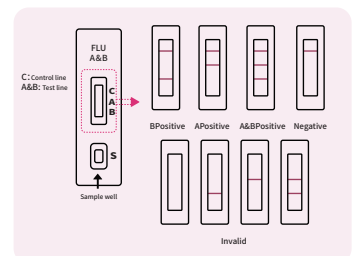

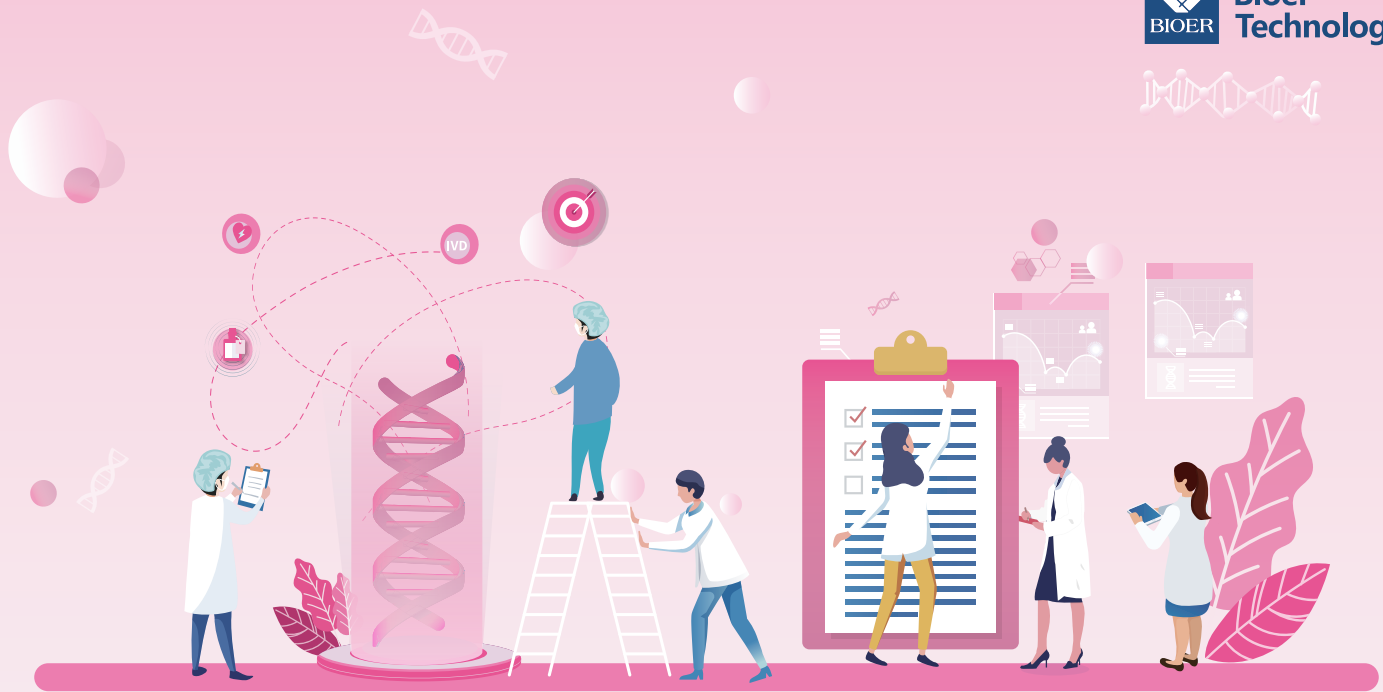

## Specifications

[www.bioer.com.cn](http://www.bioer.com.cn)

Components	Test card/Sample diluent/Sterile sampling swab
Pack Size	1 Tests/Kit; 25 Tests/Kit; 50 Tests/Kit
Storage Conditions	2°C~30°C; Valid for 24 months
Test Samples	Nasopharyngeal swabs & Oropharyngeal swabs
Reaction Conditions	15~30°C 15~20 minutes
Special Instrument	No need

## Ordering Information

[www.bioer.com.cn](http://www.bioer.com.cn)

Product Name	Cat#	Package	Sample Diluent	Sample Type
Influenza A & Influenza B Antigen Rapid Test Kit (Colloidal Gold Method)	BSK09T1S	1 Test/Kit	0.5 mL/Tube	Nasopharyngeal swabs & Oropharyngeal swabs
	BSK09S1S	25 Tests/Kit	25 × 0.5 mL/Tube	
	BSK09M1S	50 Tests/Kit	50 × 0.5 mL/Tube	



**HANGZHOU BIOER  
TECHNOLOGY CO., LTD.**

**Address:** 1192 Bin'an Road, Hi-tech (Binjiang) District, Hangzhou, 31005, P.R.China  
**Direct Phone:** +86-571-87774575 (Overseas Department) **Fax:** +86-571-87774553  
**Website:** <http://www.bioer.com.cn> **Email:** [overseas@bioer.com.cn](mailto:overseas@bioer.com.cn) **E-Date:** 2022.10

\* All rights reserved BIO K09-2210